
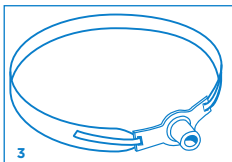


Dale® One Piece Tracheostomy Tube Holder

Application Instructions

1. Center holder behind neck.
2. Pull fastener tabs and neckband through flanges evenly until proper fit is determined.
3. Secure fastener tabs to the neckband.

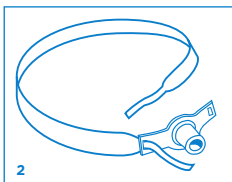

Caution

Use in supervised care settings only. Monitor to prevent unintentional disengagement. Do not trim fastener hook tabs.

FOR SINGLE PATIENT USE.

Change holder daily or more frequently as required.
Change after patient bathing.
Disposable.

Ordering Information

200	Dale® One Piece 2.54cm(1") wide, fits 24.1cm-45.5cm (9.5"-18") neck	30/Box
240	Dale® 240 Blue® 2.54cm(1") wide, fits up to 49.5cm(19.5") neck	10/Box
241	Dale® 241 PediPrints® 2.54cm(1") wide, fits up to 45.5cm(18") neck	10/Box
242	Dale® 242 PediPrints® 1.91cm(3/4") wide, fits up to 23cm(9") neck	10/Box
244	Dale® 244 Blue® 3.81cm(1 1/2") wide, fits up to 71cm(28") neck	10/Box

Individually packaged.

Dale medical products are available from your preferred hospital/medical supply dealer.

HCPCS Code: A7526

Not made with natural rubber latex.

Dale®, 240 Blue®, 244 Blue®, & PediPrints® are registered trademarks of Dale Medical Products, Inc. in the USA & EU.

Dale®



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

